Instructions for use

- *Combined CE and Operation Manual*

Instructions for Use – GB
Manual de instrucciones – ES
Instructions d'utilisation – FR
Bedienungsanleitung – DE
Istruzioni per l'uso – IT
Instruções de utilização – PT
Brugsanvisning – DK

VF405 Video Frenzel

80705351 ver. 07/2009

Instructions for use GB

VF405 Video Frenzel

Intended Use

The VF 405 Video Frenzel systems provide information to assist in the evaluation, diagnosis and documentation of vestibular disorders.

Eye movements are recorded by use of a goggle mounted with infrared video cameras.

The eye images are displayed in real time in large format on a computer monitor.

This information can then be used by a trained medical professional to diagnose vestibular disorders.

Safety Notes

 WARNING	WARNING indicates a hazardous situation which, if not avoided, could result in death or serious injury.
 CAUTION	CAUTION , used with the safety alert symbol, indicates a hazardous situation which, if not avoided, could result in minor or moderate injury.
 NOTICE	NOTICE is used to address practices not related to personal injury

Use this device only as described in this manual.

Use only the disposable Interacoustics Foam Face Cushions designed for use with the goggle.

Be sure to use only stimulation movements that are acceptable to the patient.

Equipment not complying with IEC 60601 shall be kept outside patient environment, as defined in the standard (at least 1.5 m from the patient).

Any person who connects external equipment created a system and is therefore responsible for the system complying with the requirements of IEC 60601-1-1. If in doubt, contact your service technician or local representative for help.

In order to maintain a high level of safety it is necessary to have the instrument and its power supply checked according to the medical electrical safety standard IEC 60601-1 on a yearly basis by a qualified service technician.

Exchange the disposable goggle foam cushion between patients to avoid cross contamination. Never attempt to wash the disposable goggle foam cushions

Clean the camera lens and the infrared coated mirrors of the goggle regularly to avoid shadows on the displayed images.

Do not drop and avoid other undue impacts to this device. If the instrument is dropped or otherwise damaged, return it to the manufacturer for repair and/or calibration. Do not use the instrument if any damage is suspected.

NOTICE

Although the instrument fulfils the relevant EMC requirements, precautions should be taken to avoid unnecessary exposure to electromagnetic fields, e.g. from mobile phones etc. If the device is used adjacent to other equipment caution must be taken to observe that no mutual disturbance appears.

NOTICE

Within the European Union it is illegal to dispose of electrical and electronic waste as unsorted municipal waste. Electrical and electronic waste may contain hazardous substances and therefore has to be collected separately. Such products will be marked with the crossed-out wheellie bin shown below. User cooperation is important in order to ensure a high level of reuse and recycling of electrical and electronic waste. Failure to recycle such waste products in an appropriate way may endanger the environment and consequently the health of human beings.

Warranty

INTERACOUSTICS guarantees that:

- The VF405 Video Frenzel is free from defects in material and workmanship under normal use and service for a period of 24 months from the date of delivery by Interacoustics to the first purchaser.
- Accessories are free from defects in material and workmanship under normal use and service for a period of ninety (90) days from the date of delivery by Interacoustics to the first purchaser.

If any product requires service during the applicable warranty period, the purchaser should communicate directly with the local Interacoustics service centre to determine the appropriate repair facility. Repair or replacement will be carried out at Interacoustics' expense, subject to the terms of this warranty. The product requiring service should be returned promptly, properly packed, and postage prepaid. Loss or damage in return shipment to Interacoustics shall be at purchaser's risk.

In no event shall Interacoustics be liable for any incidental, indirect or consequential damages in connection with the purchase or use of any Interacoustics product.

This shall apply solely to the original purchaser. This warranty shall not apply to any subsequent owner or holder of the product. Furthermore, this warranty shall not apply to, and Interacoustics shall not be responsible for, any loss arising in connection with the purchase or use of any Interacoustics product that has been:

- repaired by anyone other than an authorized Interacoustics service representative;
- altered in any way so as, in Interacoustics judgment, to affect its stability or reliability;
- subject to misuse or negligence or accident, or which has had the serial or lot number altered, defaced or removed; or
- improperly maintained or used in any manner other than in accordance with the instructions furnished by Interacoustics.

This warranty is in lieu of all other warranties, express or implied, and of all other obligations or liabilities of Interacoustics, and Interacoustics does not give or grant, directly or indirectly, the authority to any representative or other person to assume on behalf of Interacoustics any other liability in connection with the sale of Interacoustics products.

INTERACOUSTICS DISCLAIMS ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING ANY WARRANTY OF MERCHANTABILITY OR FOR FUNCTION OF FITNESS FOR A PARTICULAR PURPOSE OR APPLICATION.

General maintenance procedures

The performance and safety of the instrument will be kept if the following recommendations for care and maintenance are observed:

- 1 It is recommended to let the instrument go through at least one annual overhaul, to ensure that the electrical and mechanical properties are correct. This should be carried out by an authorized workshop in order to guarantee proper service and repair.
- 2 Before connection to the mains network, be sure that the local mains voltage corresponds to the voltage labeled on the instrument. Always disconnect the power cord if the instrument is opened or by inspection / replacement of the mains fuses.
- 3 Observe that no damage is present to the insulation of the mains cable or the connectors and that it is not exposed to any kind of mechanical load that could involve damage.
- 4 For maximum electrical safety, turn off the power from a mains powered instrument when it is left unused.
- 5 Do not place the instrument next to a heat source of any kind, and allow sufficient space around the instrument to ensure proper ventilation.
- 6 To ensure that the reliability of the instrument is maintained, we recommend that the operator at short intervals, for instance once a day, performs a test on a person with known data. This person could be the operator him/herself.
- 7 If the surface of the instrument or parts of it is contaminated, it can be cleaned using a soft cloth moistened with a mild solution of water and detergent or similar. The use of organic solvents and aromatic oils must be avoided. Always disconnect the mains conductor and batteries during the cleaning process, and be careful that no fluid is entering the inside of the instrument or the accessories.
- 8 After each examination of a patient, it should be ensured that there is no contamination on the parts in connection with the patient. General precautions must be observed in order to avoid disease from one patient being passed to others. The use of organic solvents and aromatic oils must be avoided.

Unpacking and inspection

Check the box and its contents for damage

When the instrument is received, please check the shipping box for rough handling and damage. If the box is damaged it should be kept until the contents of the shipment have been checked mechanically and electrically. If the instrument is faulty please contact your local distributor. Keep the shipping material for the carrier's inspection and insurance claim

Keep the carton for future shipment

The VF405 Video Frenzel comes in its own shipping carton, which is specially designed for this system. Please keep the carton. It will be needed if the system has to be returned for service. If service is required please contact your local distributor.

Reporting damage

Inspect before connection

Prior to connecting the product to the mains it should once more be inspected for damage. All of the goggle and the accessories should be checked visually for scratches and missing parts.

Report immediately any faults

Any missing part or malfunction should be reported immediately to the supplier of the instrument together with the invoice, serial number, and a detailed report of the problem. In the back of this manual you will find a "Return Report" where you can describe the problem.

Please use the "Return Report"

Please realize that if the service engineer does not know what problem to look for he may not find it, so completion of the Return Report will be of great help to us and is your best guarantee that the problem will be corrected to your satisfaction.

Concerning maintenance and repair

Interacoustics is only considered to be responsible for the validity of the CE marking, effects on safety, reliability and performance of the equipment if:

- assembly operations, extensions, readjustments, modifications or repairs are carried out by authorized persons,
- a 1 year service interval is maintained
- the electrical installation of the relevant room complies with the appropriate requirements, and
- the equipment is used by authorized personnel in accordance with the documentation supplied by Interacoustics.

It is important that the customer (agent) fills out the RETURN REPORT every time a problem arises and sends it to Interacoustics, Drejervaenget 8, DK-5610 Assens, Denmark. This should also be done every time an instrument is returned to Interacoustics. (This of course also applies in the unthinkable worst case of death or serious deterioration to patient or user).

VF405 System Configurations

The VF405 system is available in 4 different configurations:

1. VF405m Standard: one camera, eye images are only displayed – not stored for later replay
2. VF405b Standard: two cameras, eye images are only displayed – not stored for later replay
3. VF405m Extended: one camera, eye images are displayed and stored for later replay. With option of recording external situation also.
4. VF405b Extended: two cameras, eye images are displayed and stored for later replay. With option of recording external situation also.

**System contents**

Depending on system configuration the following components will be included:

Item		VF405m Standard	VF405b Standard	VF405m Extended	VF405b Extended
80481101	Camera FireWire® complete	1	2	1	2
80480502	Combi Mask VOG	1	1	1	1
80427502	FireWire® SDM express card 2P	1	1	1	1
50002501	Power supply 10W	1	1	1	1
40005901	FireWire® PCI-Card IEEE1394	1	1	1	1
76505801	EX-10000 Express card kit	1	1	1	1
36104701	Cable fastener	3	3	3	3
8502401	Disposable goggle foam pads, 24 pcs.	1	1	1	1
80427601	Foot pedal VEC	0	0	1	1
80491801	Screwdriver SE 1.5x60mm 105	1	1	1	1
	Web camera w. microphone	0	0	1	1
	Installation sheet	1	1	1	1
	Operation Manual	1	1	1	1
	CE-Manual	1	1	1	1
	Installation CD	1	1	1	1
	OtoAccess™ database installation CD	1	1	1	1
	OtoAccess™ Language Pack CD	1	1	1	1

License overview

Depending on type of license you have ordered the following functions will be available in your VF405 system:

Feature	VF405m Standard	VF405b Standard	VF405m Extended	VF405b Extended
OtoAccess™ patient database	X	X	X	X
Toggle full screen display			X	X
Video storage			X	X
Video play back			X	X
Elapsed test time			X	X
Report editor	X	X	X	X
Sound recording			X	X
Test title creator			X	X
Footswitch operated video capture			X	X
Footswitch operated fixation light			X	X
Toggle one camera/two cameras		X		X
SW controlled Fixation light	X	X	X	X
External camera enabled			X	X
Toggle picture quality high/normal	X	X	X	X

System Requirements

Hardware requirements

Laptop PC

- Intel Core 2 Duo processor 1.8 GHz or better
- Minimum 1 GB of RAM
- One 34 mm PCExpressCard slot available
- One free USB port available for footswitch
- One free USB port available for external Web camera
- Monitor resolution 1024 x 768 or better

Desktop PC

- Intel Core 2 Duo processor 1.8 GHz or better
- Minimum 1 GB of RAM
- One free slot available for IEEE1394a FireWire® PCI board
- One free USB port available for footswitch
- One free USB port available for external Web camera
- Monitor resolution 1024 x 768 or better

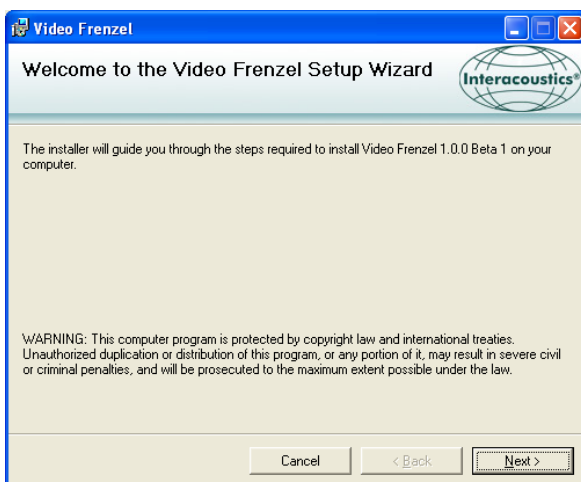
Software requirements

- Windows®XP SP2 Operating System, **or**
- Windows®Vista Business SP1, **or**
- Windows®Vista Ultimate with or without SP1
- Microsoft .Net 2.0 Framework
- Windows Installer 3.x


FireWire and the FireWire symbol are trademarks of Apple Computer, Inc., registered in the US and other countries. The FireWire logo is trademark of Apple Computer Inc.

Installation


To install the Video Frenzel you need to insert the installation CD and follow the steps in the installation procedure. To initiate the installation procedure manually, click “Start”, then go to “My Computer” and double-click the DVD/CD-RW Drive to view the contents of the installation CD. Double click the “setup.exe” file to initiate the installation.

Click “Next”.

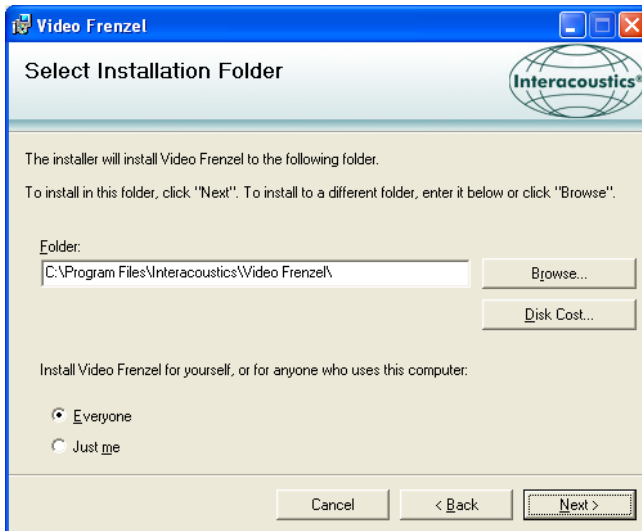Click "Next".

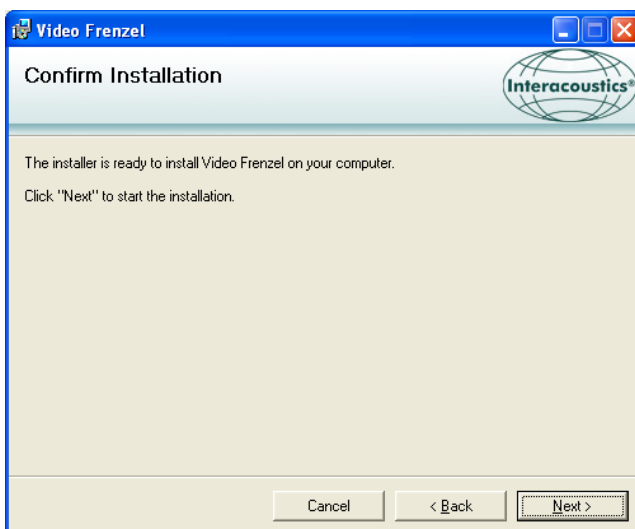
Click "Next" and wait while installing.

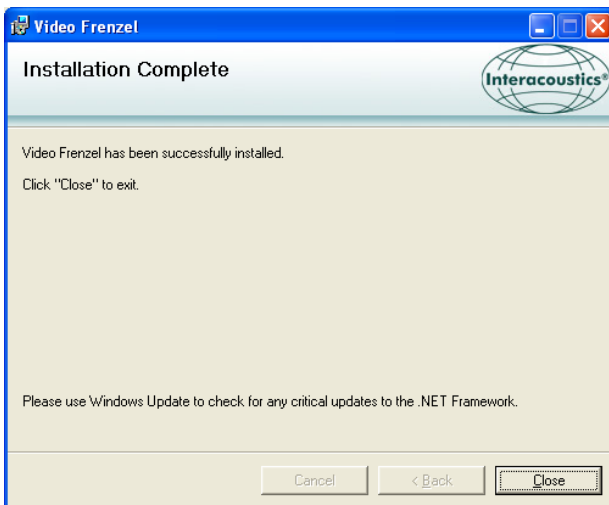
Click "Continue Anyway"

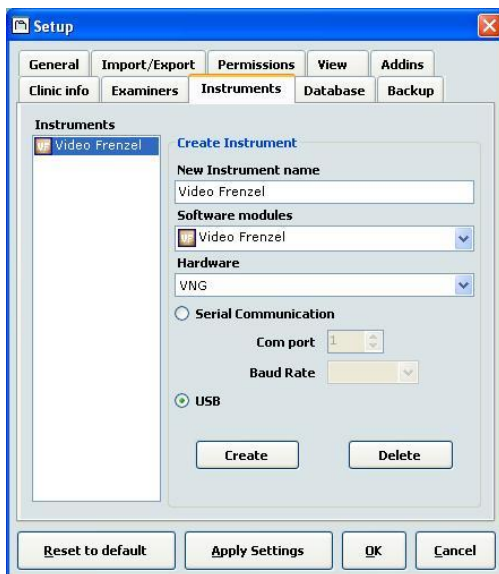Click "Close". Video Frenzel is now installed. Now you need to setup OtoAccess™ · Start OtoAccess™ and enter File->setup->Instruments. Create a new "Video Frenzel" instrument as shown below


To start Video Frenzel double-click "Video Frenzel" in the instruments list

License setup


Each of your Video Frenzel systems has a unique License Key, which has to be entered in "License Setup" before the system functions.

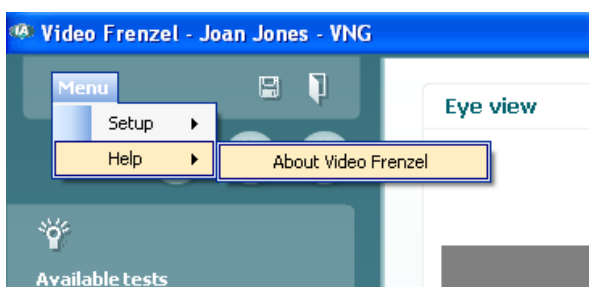In order to install the license/licenses the camera/cameras must be correctly connected to the computer. Having done this select 'Menu' → 'Help' → 'About Video Frenzel' in the VF405 user interface:



Then you click "License".

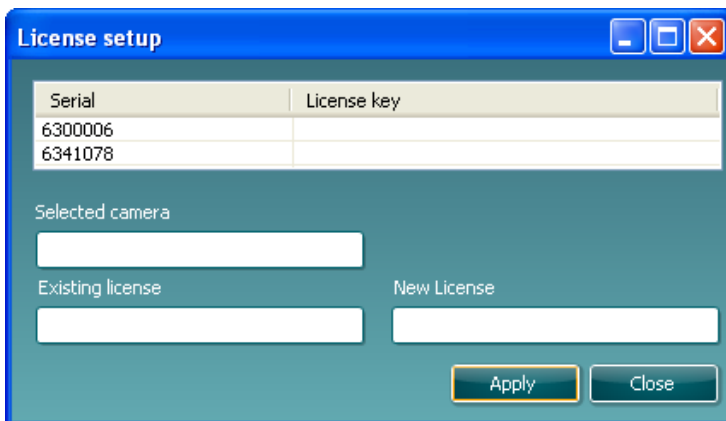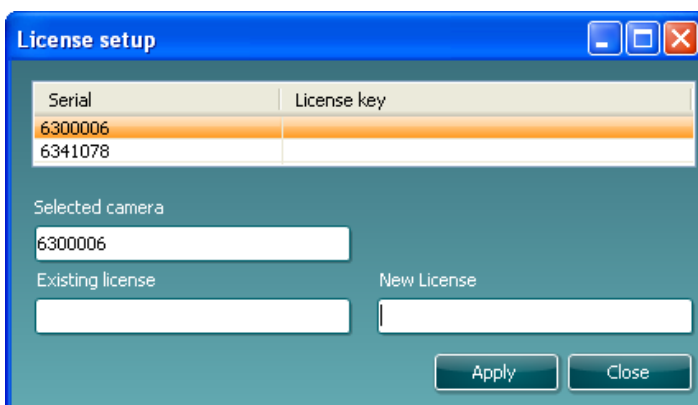The serial numbers listed will be different depending on the cameras connected to the computer. To enter a license you select the desired camera by placing the cursor on the serial number and left click the mouse:

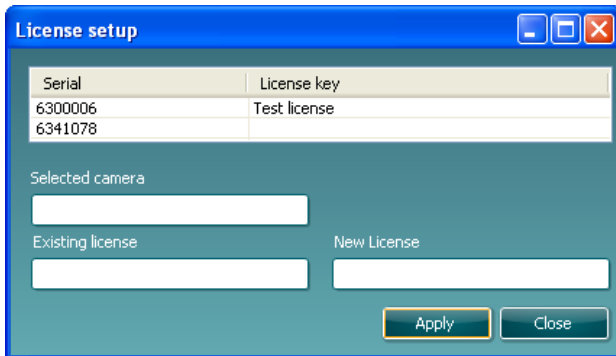
The camera licences in question appear from the enclosed "License Certificate". The license is a combination of 23 figures and letters to be entered in the "New License" box. After that you click "Apply" to save license. The license can be entered by random use of small or capital letters.


When license for both cameras has been entered you click "Close".

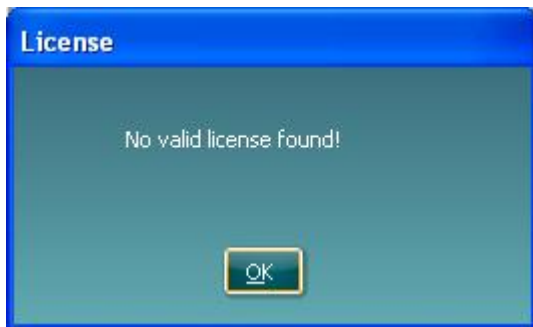If you have entered an invalid license you will be presented with this error message and Video Frenzel functionality will be disabled until you enter a valid license again:

Setting up Video Frenzel

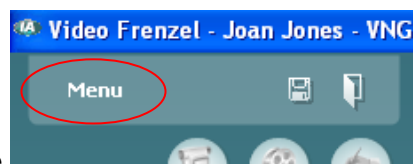
To enter setup click the  icon in the top menu.

FireWire® Camera setup

Video Frenzel will only work with FireWire® cameras.

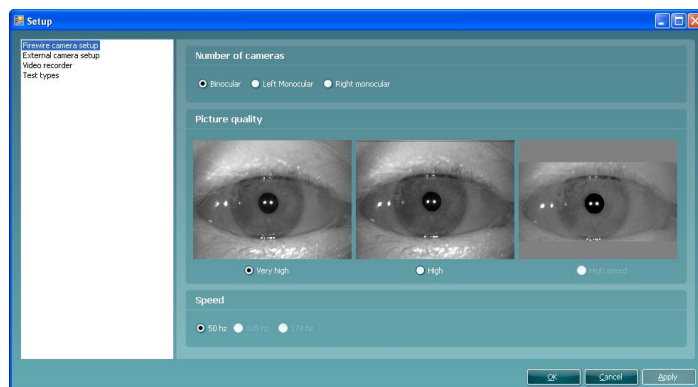To change the camera setup from binocular mode to monocular mode or vice versa, simply click the relevant configuration in the 'Number of Cameras' dialogue


External camera setup (only VF405 Extended)

The VF405 Extended versions include an autofocus web camera with built-in microphone for recording of video and sound from the examination procedures. Any USB web camera that support DirectShow can be used if the supplied web camera does not meet the requirement of the user.

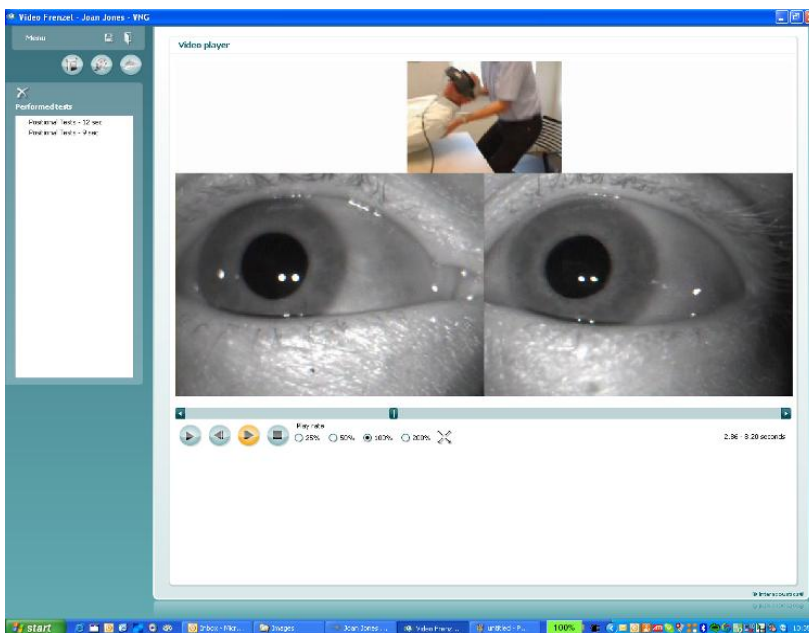The recordings from the external camera are synchronized with the recordings from the infrared cameras, and thereby enables synchronized, simultaneous replay of both the eye movements (from the infrared cameras) and the test procedures (from the external camera). This allows the examiner to observe the eye movements as a result of a given positional/positioning stimulus:


Installation of the included Logitech® QuickCam® web camera.


NOTICE

You must install the Logitech®QuickCam® software before connecting your camera, or some of the camera's features may not function properly. Only connect your camera when prompted to do so during the software installation process. You can obtain installation instructions and software drivers at the following website: www.logitech.com/support

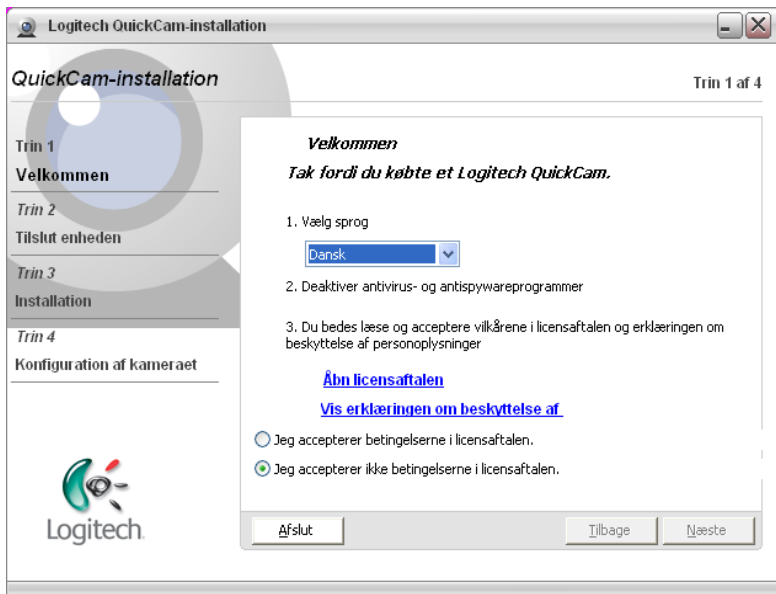1. Installing the Logitech®QuickCam® software:
 - a. Insert the installation CD into your CD-ROM/DVD drive. Your CD should start automatically. If it does not, follow the steps below:
 - i. Double-click the My computer icon. Alternatively, click Start ► My computer
 - ii. Right-click the CD-ROM/DVD-ROM drive icon, and then click Autoplay
 - b. The following dialogue appears:

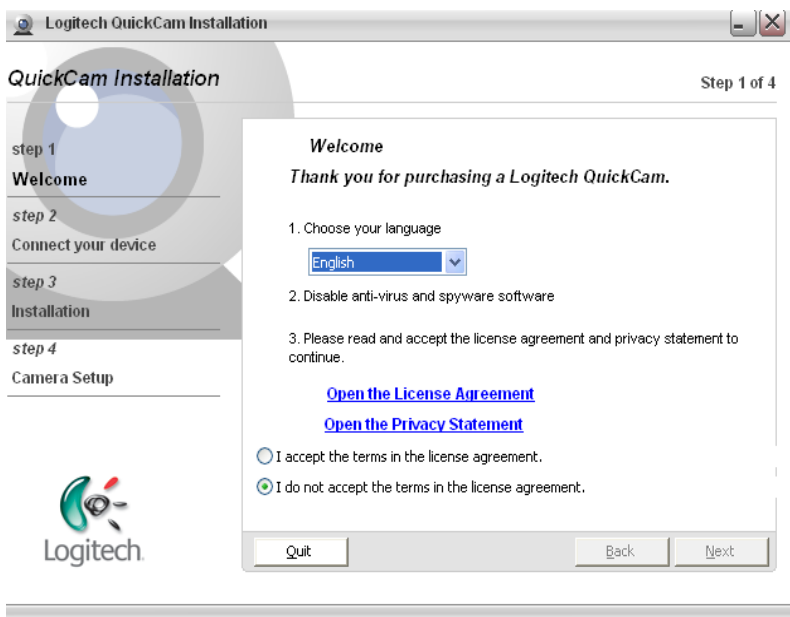
- c. To change the language, open the pull-down menu under “Vælg sprog” and select the desired language – in this case English

- d. Click “I accept the terms in the license agreement”, then click the “Next” button
2. Connect Your Camera to Your Computer
 - a. Wait until you are prompted by the software before connecting your camera to an available USB port on your computer.

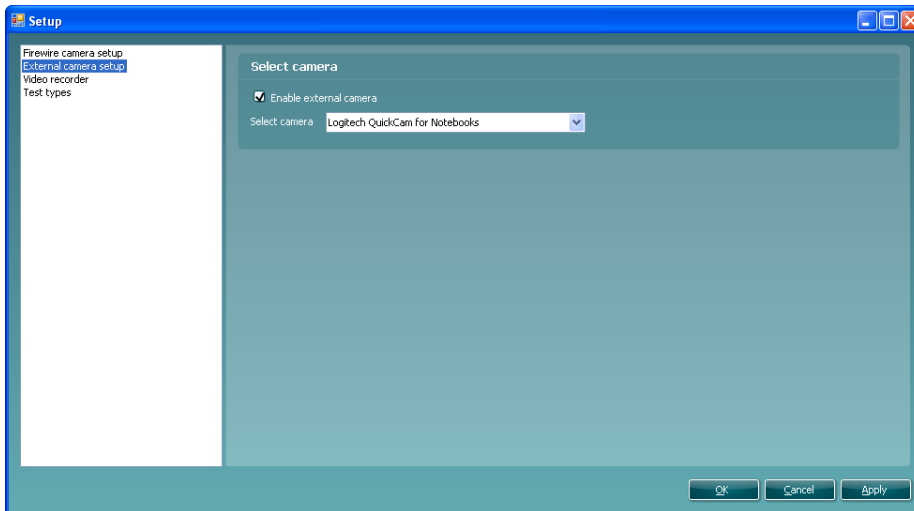Once the web camera is installed you can enable it in Video Frenzel.
Frame rate of web camera is fixed to 15fps, to spare system resources.


1. To enable the external camera check the “Enable external camera” check box.
2. Select you camera in the drop down below.
3. Press “Ok” to return to Video Frenzel. The external camera image will be shown above the normal eye images. If the external camera is enabled then it will always be recorded together with the eye images.


Installation of the Footswitch (only VF405 Extended)

Connect the footswitch to any available USB port on your computer.

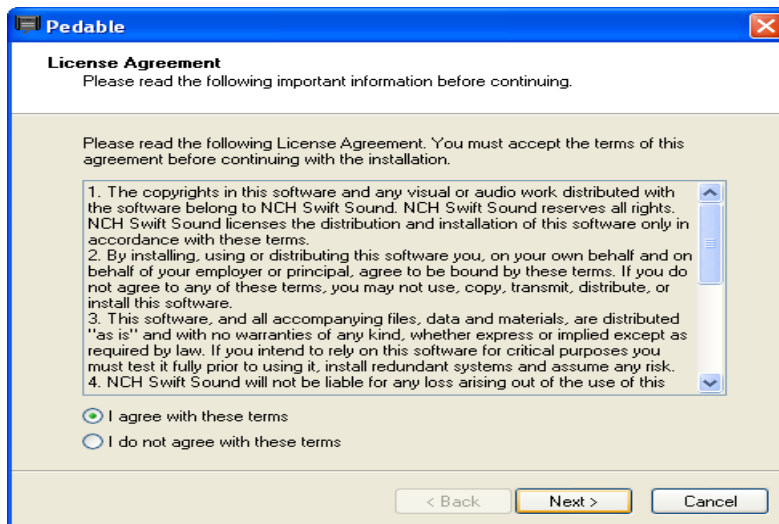Go to the Video Frenzel installation CD and locate the folder”Foot pedal setup”. Start the program “Pedablesetup.exe”.

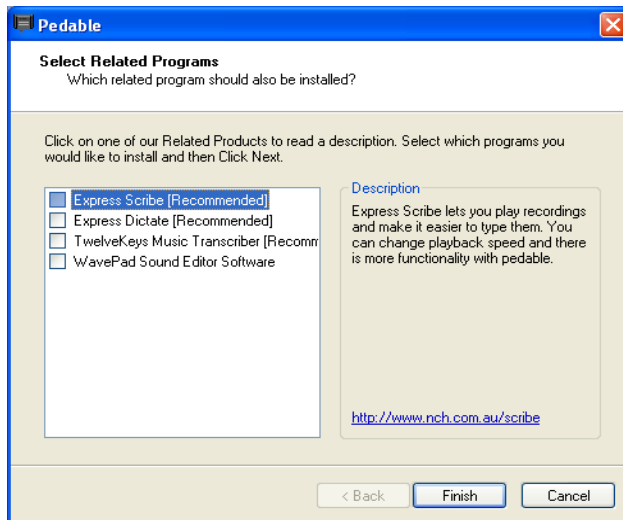
Select “I agree with these terms” and click Next. Now the program will install.


Deselect all options by removing the ticks and click Finish.

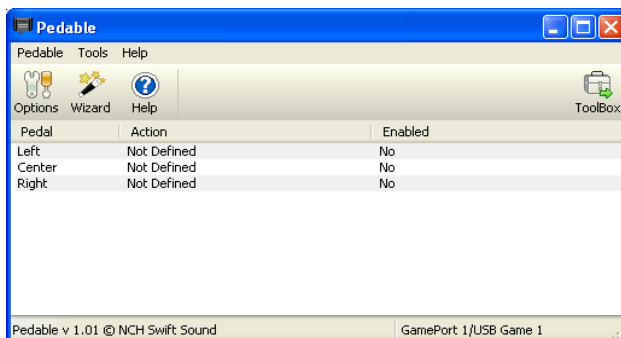
Now the Pedable program will start automatically. If not you can start the program manually from the Start menu->All Programs->Pedable->Pedable

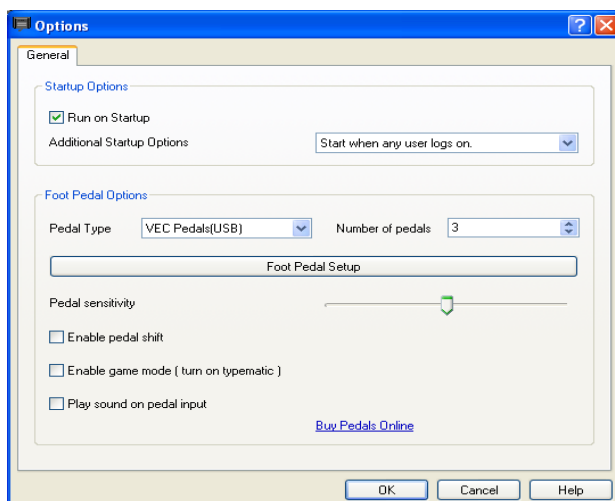
Click Options and make sure setup details are exactly like below:

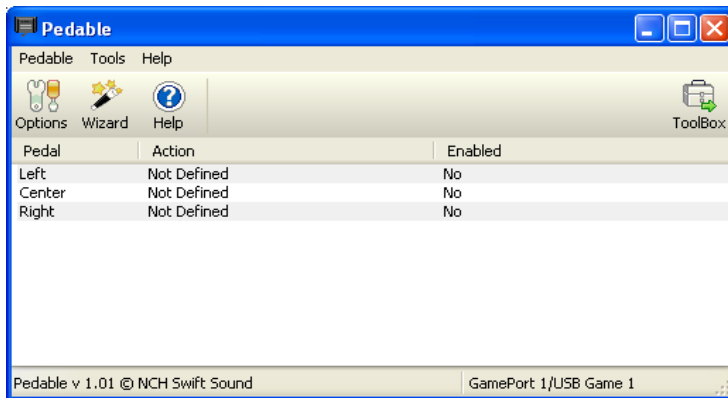
Click OK and return to main screen. Now double-click the "Left" entry to setup this button. The function is "Shift between video on full mode and ordinary user interface".


The following screen appears:

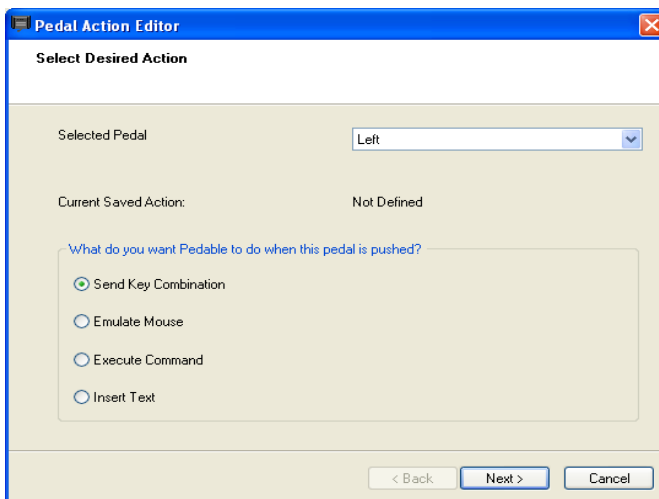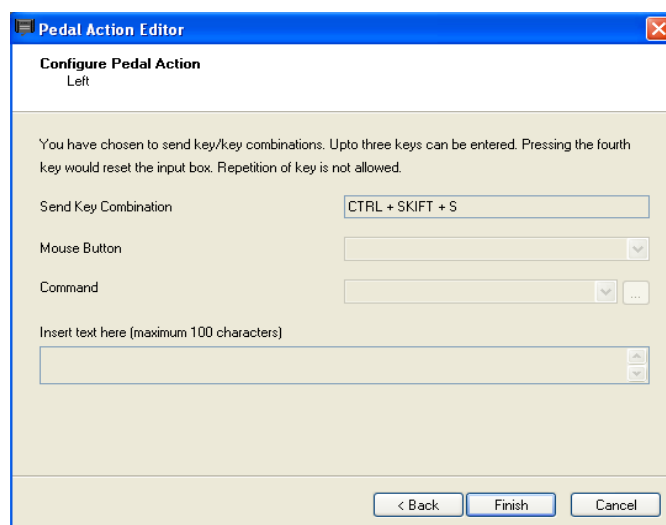
Select "Send Key Combination" and click Next.

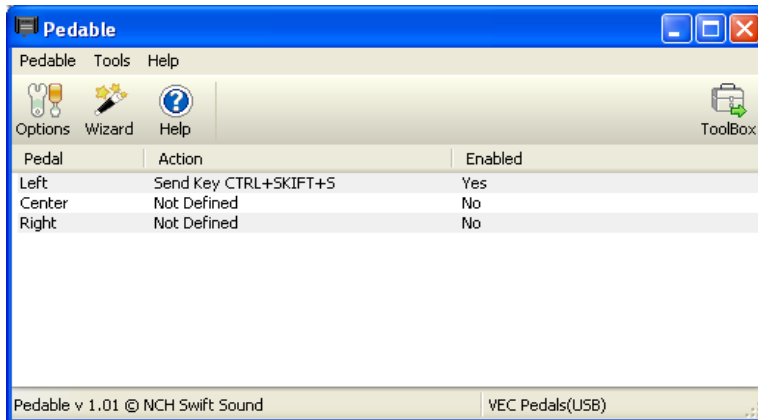
Press CTRL + SHIFT + S simultaneously to get the above screen result. Click Finish.

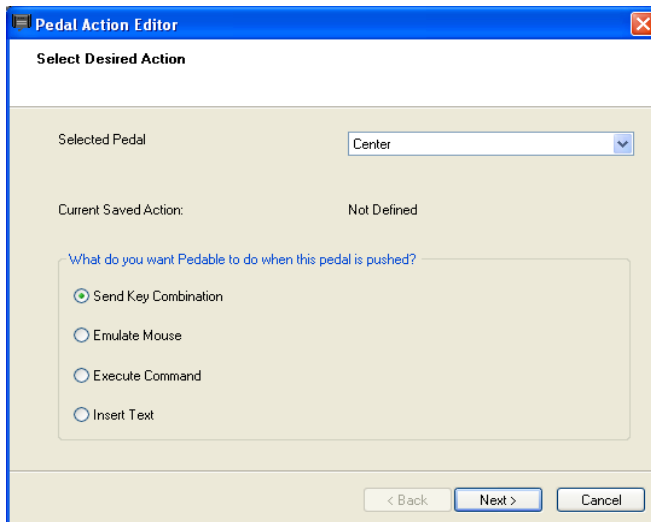
Double-click the Center entry to setup the center pedal. The function is "Start/stop recoding sound and video".

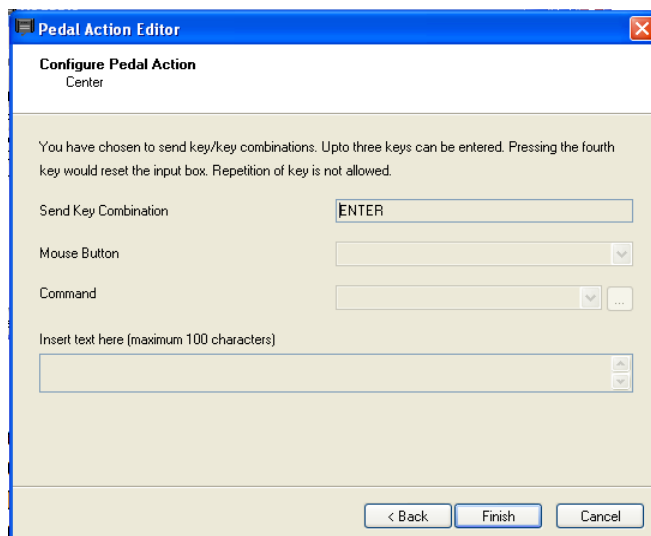
Select "Send Key Combination" and click Next.

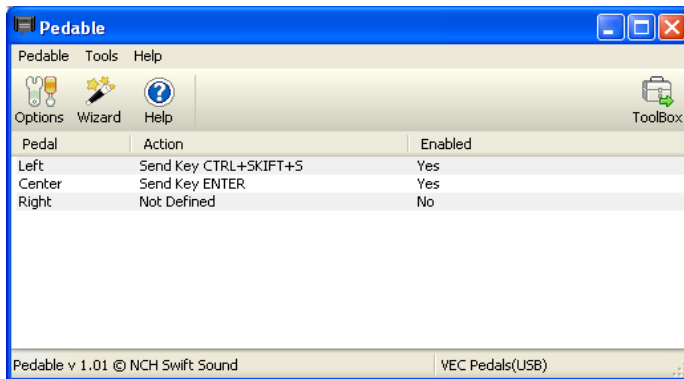
Press Enter to get the above screen result. Click Finish.

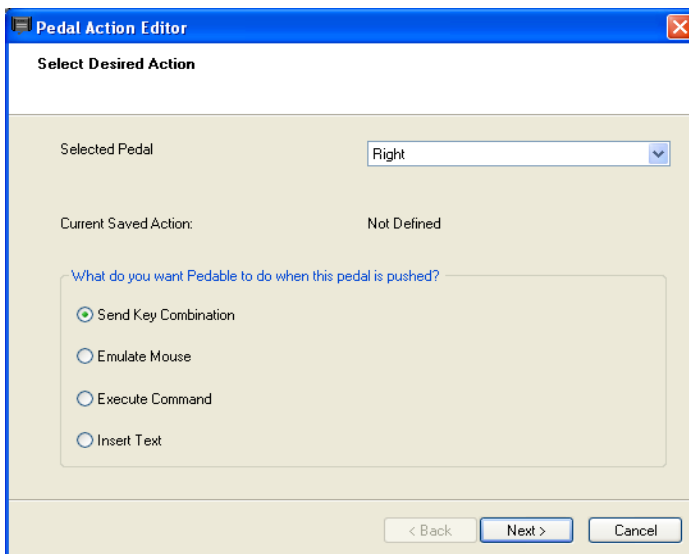Double-click the Right entry to setup the right pedal. Function is "Turn on/off fixation light in goggles".

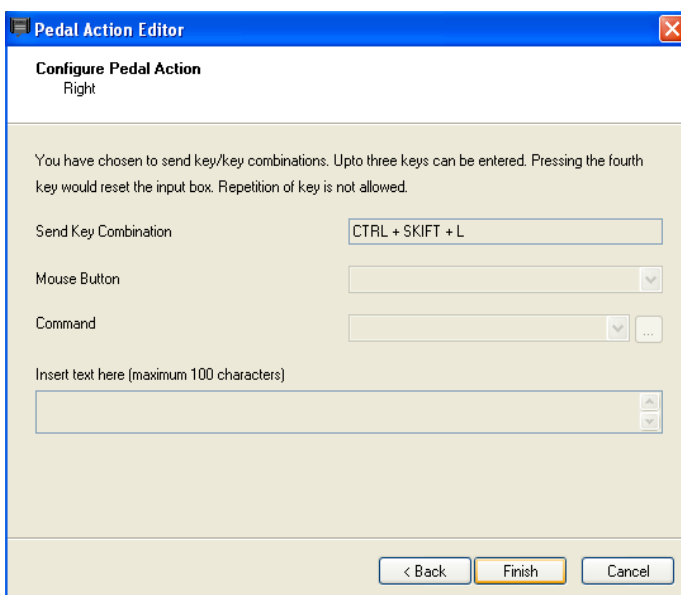Select "Send Key Combination" and click Next.


Press CTRL + SHIFT + L simultaneously to get the above screen result. Press Finish. Setup is now complete and you can close the Pedable setup program. The Pedable software must however always be running in the lower right corner for the foot pedal to work correctly.

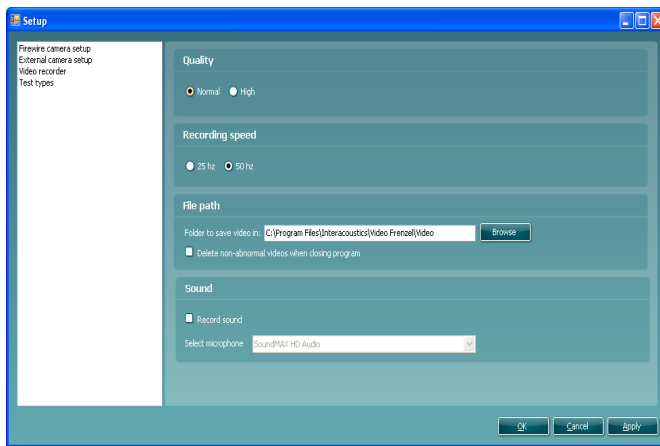Video recorder setup (only VF405 Extended)


The quality section allows you to select quality of video. Setting should be left on “Normal” as this uses lesser computer resources. If you want video to be at best quality you can select “High” quality – this is very computer demanding.

The recording speed section allows you to select the number of frames per seconds that is written to the video. The higher the setting the smoother the video will look when the eye moves rapidly. Again the higher the setting the more computer resources is needed. 25 or 50 Hz is usually enough.

The “Same as camera” option can be selected if you always want to record at the same frame rate as the cameras. This allows you to record at e.g. 50 frames per second.

The file path section allows you to specify where to store the video recorded. If you check the checkbox “Delete non-abnormal videos when closing program”, then Video Frenzel will automatically delete videos that are not marked with a triangle symbol  as interesting in the video player when the program is closed.

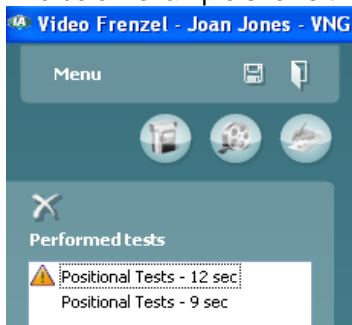The below example shows that it is only "Positional Tests - 12 sec" that will be saved.


The sound section allows you to enable sound recording. To record sound you enable the “Record sound” checkbox. Select the desired microphone from the list below.

Test types setup (only VF405 Extended)

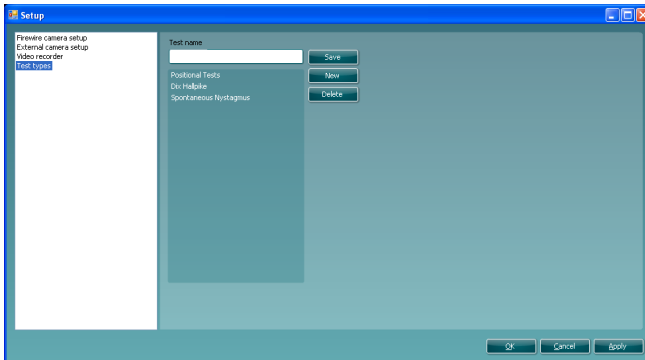
This screen allows you to setup test types available for recording.

To create a new test type you enter a test name in the “Test name” box and click save. To delete a test type you select the desired test type and click delete.

Video Recording (only VF405 Extended)

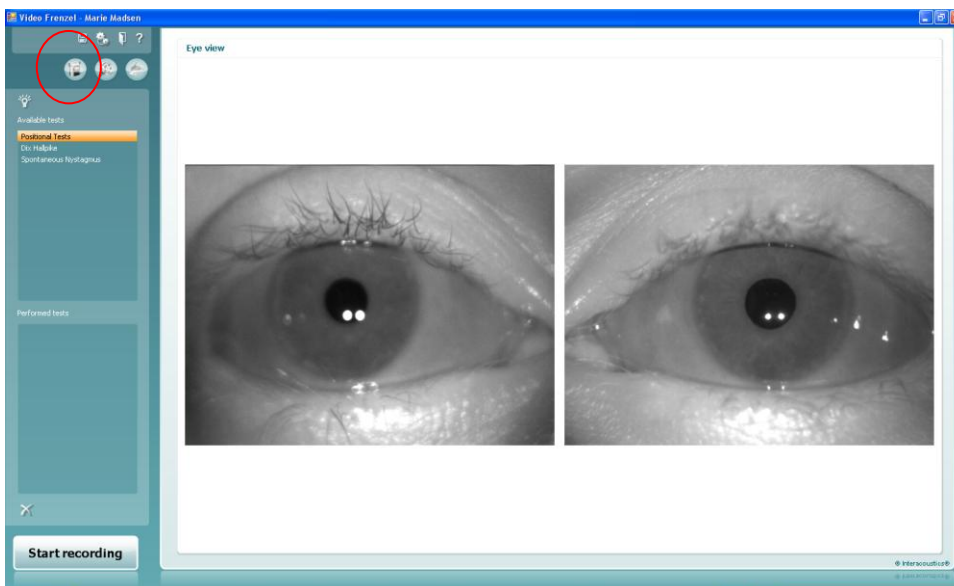This is the correct screen for video recording. You can access the screen by clicking on the first icon in the row.

1. To record video you first need to select the test you wish to perform. You do this by selecting a test in the “Available tests” list.
2. Next you click “Start recording” to start recording video.
3. If you wish to view the eyes in full screen mode while recording you simply double click one of the eyes. Double click on one of the eyes again to exit full screen mode.
4. Click “Stop recording” to stop recording again. When you stop recording, then an entry will be added to the “Performed tests” lists. This list contains recorded sessions. If you wish to delete a video you select the test in performed tests list and click the cross below the list.
5. To enable or disable fixation light during recording click the  button.

Foot pedal usage

It is possible to use the USB foot pedal while recording video. The buttons have the following functionality:

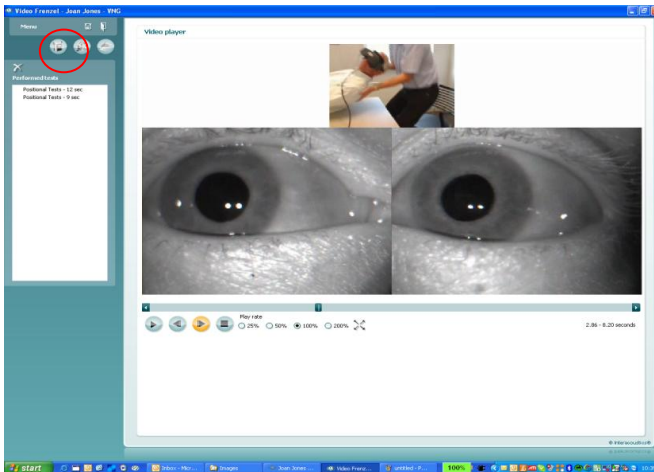
Left button: Toggle full screen mode.

Center button: Start/stop recording.

Right button: Toggle fixation light

Playback video (only VF405 Extended)

To play back video you need to go to the second icon in the row – click on the “Open video player” icon in the left top of the screen.

1. To play a certain test you select it in the “Performed tests” list.

2. To start video click the  button. Once clicked it will change to a pause button.

3. To pause the video click the  button.

4. To step one frame backwards click the  button.

5. To step one frame forwards click the  button.

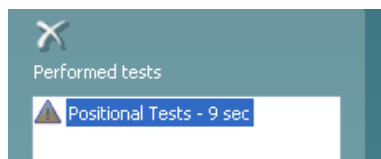
6. To stop the video click the  button

7. If you wish to playback the video at a slower or faster speed select one of the radio buttons:

25% 50% 100% 200% . 100% is the normal playback rate.

8. If you wish to playback video in full screen mode click the  button. Click the same button again to exit full screen mode.

9. If you want to consider the video abnormal by means of the  symbol, click the symbol with the mouse after having placed the cursor to the left of the test name:

This will prevent the video from being deleted if the checkbox “Delete non-abnormal videos when closing program” is checked in the setup.

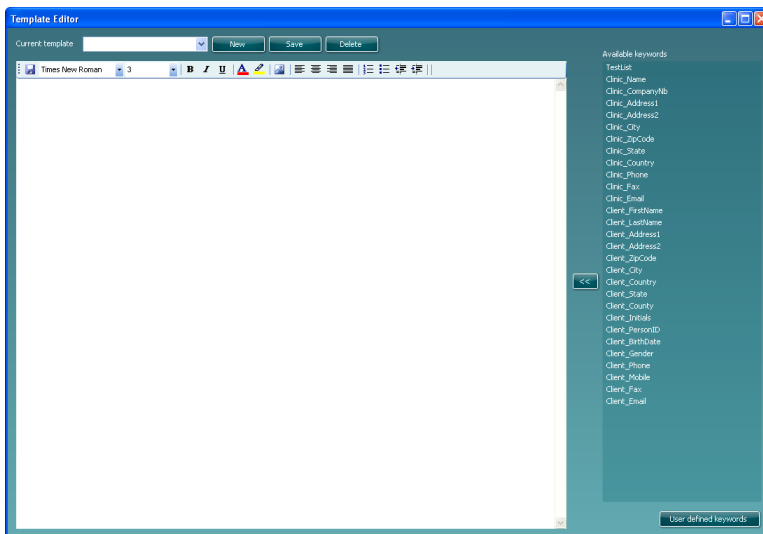
Report editor

This section allows you to create a report. Typically you would have one or more report templates available.


Creating a report template

Click the icon to open template editor.

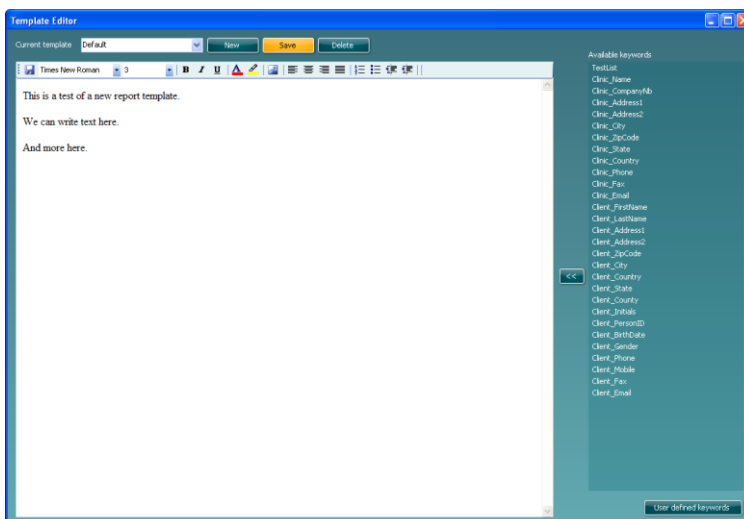In this dialog you can create or modify report templates. To create a new report template you click ‘New’ to create a blank document and enter a template name where the cursor blinks. Then you enter the text for the report template in the window below and hit ‘Save’ when done.

The list to the right contains keywords that you can insert into the template. E.g. the following two keywords “<<Client_FirstName>> <<Client_LastName>>” will be replaced with the client’s first name and last name when the report template is selected in the report editor.

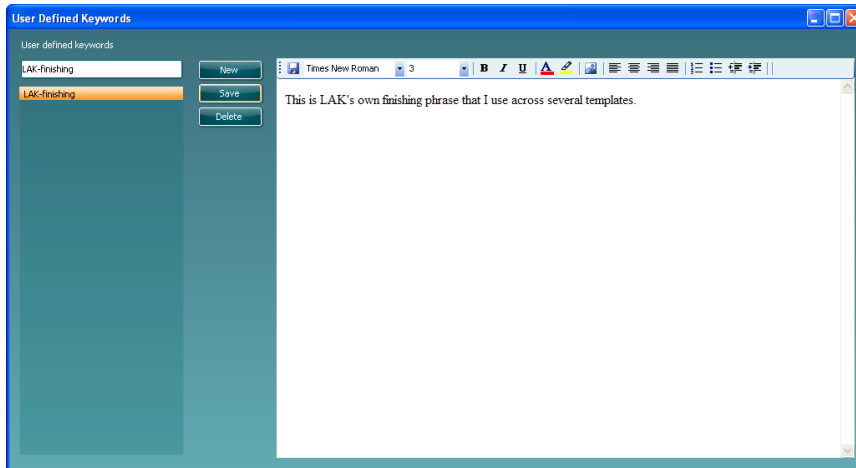
To insert a keyword you place the cursor where you want the keyword to be inserted and double-click on the keyword name in the list. Alternatively you can select the desired keyword and insert it into the report template by clicking


User defined keywords

If you have a certain phrase that you often use then you can put it into a user defined keyword. This will save you from entering the same text again and again.

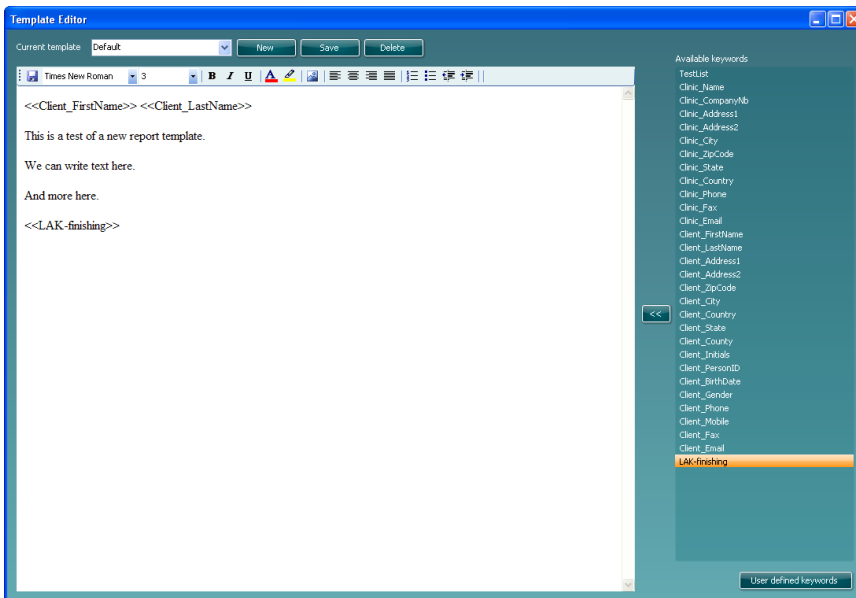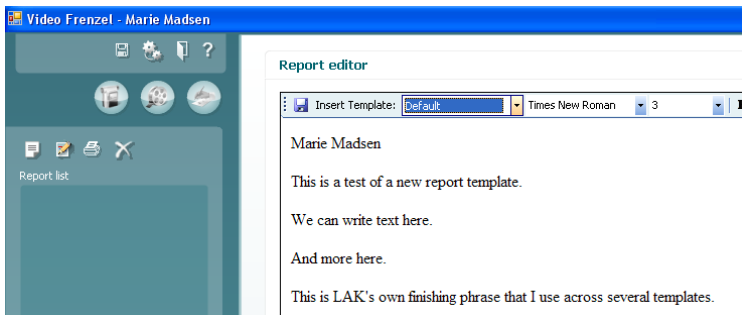
To create a new user defined keyword you click the 'New' button and enter a name. Then you enter the text or images for the keyword and click 'Save' when done. When returning to the template editor, your user defined keywords will be listed in the bottom of the keyword list.

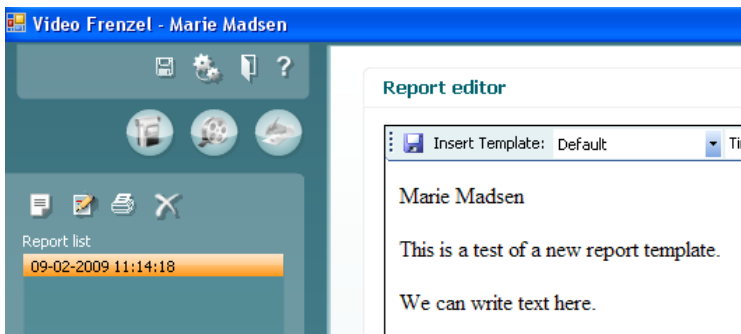
Once you have setup your report template you can return to the report editor and use your report template. To do this you simply select the name from the dropdown list like shown below:

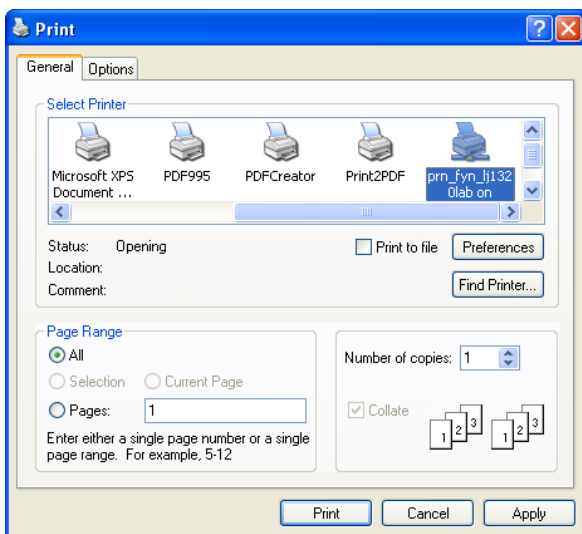
Saving a report

If you wish to save a report you click the  icon. Once saved the report's date and time will be added to the list:

Printing a report

To print a report you click the  icon. This will open a dialog asking you to select what printer to use:

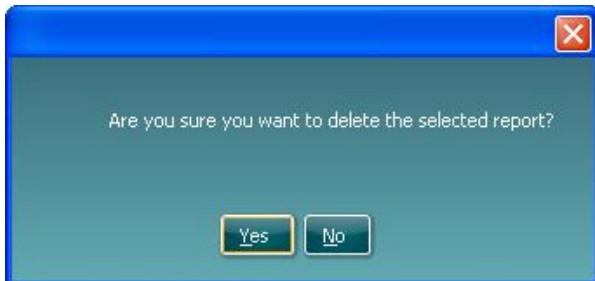
Select the desired printer and click 'Print'.

Deleting a report

Select the desired report and click the  icon.

When asked you select 'Yes' to delete the selected report.

Formatting text in report

You can insert pictures, change font type, size, colour and position via the toolbar:

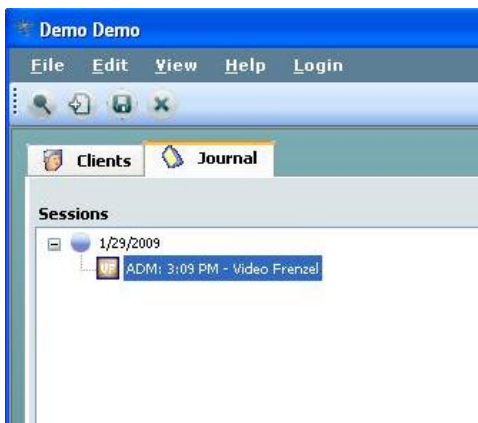
Saving data

To save the videos recorded click the disk icon  in the top toolbar

To exit and save click the  icon.

Re-opening a recorded session

To open a recorded session again you need to go to OtoAccess™ and select the "Journal" tab. Double click the desired test to open it again.

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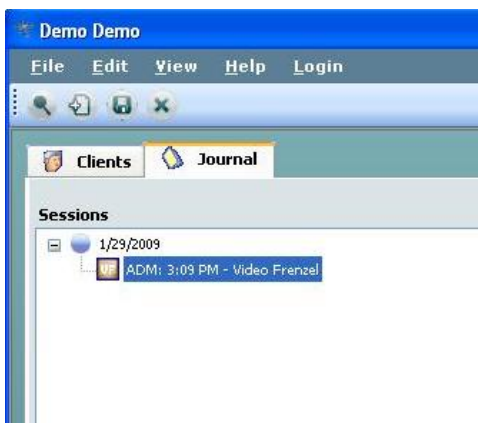
Fax
(+45) 63713522

E-mail
info@interacoustics.com

- returned to INTERACOUSTICS for: repair, exchange, other: _____
- defective as described below with request of assistance
- repaired locally as described below
- showing general problems as described below

Item: _____ Type: _____ Quantity: _____

Serial No.: _____ Supplied by: _____

Included parts: _____

Important! - Accessories used together with the item must be included if returned (e.g. external power supply, headsets, transducers and couplers).

Description of problem or the performed local repair:

Returned according to agreement with: Interacoustics, Other : _____

Date : _____ Person : _____

Please provide e-mail address or fax No. to whom Interacoustics may confirm reception of the returned goods: _____

The above mentioned item is reported to be dangerous to patient or user ¹

In order to ensure instant and effective treatment of returned goods, it is important that this form is filled in and placed together with the item.
Please note that the goods must be carefully packed, preferably in original packing, in order to avoid damage during transport. (Packing material may be ordered from Interacoustics)

¹ EC Medical Device Directive rules require immediate report to be sent, if the device by malfunction deterioration of performance or characteristics and/or by inadequacy in labelling or instructions for use, has caused or could have caused death or serious deterioration of health to patient or user.